

**LASER SAFETY SYSTEM**  
**"FOR THE SAFETY OF THE PATIENT"**  
**Patient Laser Eye Shield (Binocular)**  
U.S. Patent 4,635,625



**Indication:** This product is intended for use on the following procedure:

1. CO<sub>2</sub> Laser; @ 1/2" /50 watts/continuous beam and 60 seconds.
2. YAG Laser; @ 1/2" /70 watts/0.5 second pulsed beam.
3. ARGON Laser; @ 1/2" /14 watts/continuous beam and 60 seconds.

**Instruction for Use:**

1. Clean and dry application site as is required.
2. Open package only immediately prior to use.
3. Prior to patient application fully moisten eye gauze pads using a hospital prescribed solution. Reduce risk of poor adhesion by restricting moisture just to the eye gauze pads.
4. Prep the patient's eyes per hospital's standard procedure.
5. Remove eye shield from its backing.
6. Avoid excessive skin or finger contact with adhesive surface prior to application.
7. Place the eye shield over the eyes. Border of the eye shield should lay on the orbital ridges with the gauze pads covering the eyes. Press the perimeter of the eye shield and the border over the nose to assure fit and adherence.
8. Prior to surgical draping recheck the eye shield to assure adequate eye cover and protection.

**Warnings & Precautions**

- Laser procedure should only be performed by qualified and professional persons who have been appropriately trained in laser safety.
- Operator or personnel in the laser room must wear the specific type of Laser Safety Eyewear (LSE) for the laser wavelength at which protection is afforded.
- Do not use on patients with known allergy to EKG adhesive.
- Do not reuse. Do not relocate the eye shield after initial application to the patient.
- Do not cut or modify the eye shield.

**Caution**

Federal Law restricts this device to sale by or on the order of a physician.

Manufactured by:  
Olsen Medical  
3230 Commerce Center Place  
Louisville, KY 40211  
(502) 772-4280 <> (800) 297-6344

0008032 Rev: 3

**LIMITED WARRANTY FOR LASER MATERIAL**

Seller warrants that the protective laser material (hereinafter referred to as the "Product") delivered to buyer pursuant to this Agreement shall conform to applicable specifications and quality control standards for such Product as described herein. Seller has taken every reasonable precaution to ensure the Product will perform as an effective barrier against light reflection if used within the testing parameters as outlined herein. **THE PRODUCT IS NOT INTENDED TO PROTECT AGAINST INTENTIONAL EXPOSURES.**

Buyer assumes all risks and liability for results obtained by the **IMPROPER USE** of said Product. Except for the warranty stated herein, Seller **MAKES NO FURTHER REPRESENTATIONS OR WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING ANY WARRANTIES AS TO MERCHANTABILITY, FITNESS FOR ANY PURPOSE OTHER THAN THE SUGGESTED USE STATED HEREIN, OR ANY OTHER PURPOSE OTHER THAN THE SUGGESTED USE STATED HEREIN, OR ANY OTHER MATTER. IN NO EVENT SHALL SELLER BE LIABLE FOR SPECIAL, GENERAL, CONSEQUENTIAL, INCIDENTAL, EXEMPLARY OR SIMILAR DAMAGES.**

**Olsen MEDICAL®**

3230 Commerce Center Place, Louisville, KY 40211  
(502) 772-4280 <> (800) 297-6344