For product information, contact:
Lisätietoja tuotteesta antaa:
För produktinformation, kontakta:
Voor productinformatie, neem contact op met:
Para información sobre el producto, contacte a:
Pour obtenir des informations sur le produit, veuillez contacter :
Produktinformationen:
Per informazioni sui prodotti, contattare:

**Olsen MEDICAL**<sup>®</sup>

3230 Commerce Center Place  
Louisville, KY 40211  
(502) 772-4280 <> (800) 297-6344

**CE** 0120

Authorized European Representative:

Keaney Medical Limited  
Clinitron House  
Greenhills Business Park  
Greenhills Road, Dublin 24  
353 1 459 6585  
Made in USA

Reference number / Tuotenumero / Referensnummer / Referenznummer / Numéro de referencia / Numéro de référence / Bezugsnummer / Numero di riferimento

Lot number / Eränumero / Satsnummer / Partijnummer / Número de lote / Numéro de lot / Seriennummer / Numero di lotto

See instructions for use / Lue käyttöohjeet / Se bruksanvisningen / Zie gebruiksaanwijzing / Vea las instrucciones de uso / Voir les instructions d'utilisation / Gebrauchsanweisung beachten / Consultare le istruzioni per le modalità d'uso

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No Latex

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

## ENGLISH

### REUSABLE ELECTROSURGICAL DEVICES

**Description:** Olsen Medical reusable electrosurgical devices are sold as non-sterile, reusable, multi-use devices; unless otherwise noted. Each reusable device is designed to withstand 20 sterilization cycles when properly cared for and sterilized according to the instructions below. These devices are designed to function as the active electrodes in electrosurgical systems or as accessories to active electrodes in electrosurgical systems. This family of devices includes; but is not limited to: electrosurgical forceps, electrode handles, electrodes, cables, smoke evacuators or suction coagulators.

**Handling:**

- Inspect device before each use. Discard or send for repair if damaged.
- Device use and application to be determined by doctor.
- Begin decontamination immediately after a surgical procedure. Wipe off all debris with a moist sponge or gauze pad. Clean devices with mild detergent (Tergazyme or a similar type solution) that is safe for plastics. If necessary, except for cables, place devices in ultrasonic bath with cleaning solution for 10 minutes at 60 kHz. Rinse thoroughly with running tap water to remove any debris or cleaning solution. Wipe dry and sterilize according to the instructions below.

Note: Remove and discard the tip protector on all forceps or electrodes before use.

**Storage:** Store at +5°C to +30°C. Relative humidity: 30% – 75%, non-condensing.

**Sterility:** These devices are sold as NON-STERILE. Steam sterilize before each use.

**Prevacuum Steam Sterilization:**

- Double wrap device
- Select vacuum cycle. Set temperature to 270°F (132°C)
- Set sterilization time for three minutes and drying time for 5 minutes. Run cycle.
- Allow to cool before use.

**Caution:**

- Damaged electrosurgical devices may cause electrical burns to the patient or doctor. Inspect every device before use.
- Active electrosurgical devices must be placed away from the patient when not in use. A holster is recommended.
- During use, keep the active electrode clean; build up of desiccated tissue will cause poor performance. A tip cleaning pad is recommended.
- Use with active electrodes only in RF surgical applications. Consult RF Generator owner’s manual.
- Inspect all plugs, wires, and connections for any cuts, breaks or loose wires.
- Worn or defective cords cannot be repaired. Replace worn or defective cords to maintain optimum reliability in operating room use.
- Non-insulated switching contacts are not to touch or be in close proximity to the skin or bodily tissue to prevent patient/surgeon burn.
- Improperly seated electrode may cause burn or injury. Check that the electrode is compatible with the pencil or handle and ensure that the insulated sleeve fits securely inside the handle. Do not use electrode if it does not fit securely inside the handle.
- Do not bend, cut, or modify electrode before or during surgery. If bent, it may break the device or cause damage to insulation.
- Keep the voltage/power settings at the lowest possible level to achieve the desired effects.
- Avoid unnecessary or prolonged activation. Activate only when touching or immediately next to the point of contact.
- Protect against avoidable injury by establishing a protocol for device examination and maintenance.

**Warning:** Careless use of any element in electrosurgical systems may cause serious burn. The use of the device with safely functioning and compatible handles, cords, electrosurgical generators and accessories is the user’s responsibility. Read and understand all precautions and instructions before attempting to use any electrosurgical device.

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